

MyCare Psychiatry Total Risperidone Assay Kit

This package insert must be read carefully prior to product use. Package insert instructions must be followed accordingly. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

CUSTOMER SERVICE



Telephone: +1 610 419-6731
 Fax: +1 484 547-0590
 Email: Techsupport@saladax.com
 116 Research Dr.
 Bethlehem, PA 18015 USA
 www.saladax.com



| | | | |
|----------------|---|------------------------|------------------------------|
| IVD | <i>in vitro</i> Diagnostic Device | | Consult Instructions for Use |
| REF | Catalog Number | | Use By |
| LOT | Batch Code (Lot) | | Temperature Limitation |
| EC REP | Authorized Representative in the European Community | | Manufacturer |
| Rx only | For Prescription Use Only | R1 R2 | Reagent 1 Reagent 2 |

INTENDED USE

Rx only

The MyCare Psychiatry Total Risperidone Assay Kit is intended for the *in vitro* quantitative measurement of total risperidone (risperidone plus 9-hydroxyrisperidone) in human serum using automated clinical chemistry analyzers. Measurements obtained are used for monitoring patient adherence to risperidone therapy to help ensure appropriate treatment.

SUMMARY AND EXPLANATION OF THE TEST

Risperidone (3-[2-[4-(6-fluoro-1,2-benzisoxazol-3-yl)-1-piperidinyl]ethyl]-6,7,8,9-tetrahydro-2-methyl-4H-pyrido[1,2-a]pyrimidin-4-one) is a benzisoxazole derivative, atypical antipsychotic agent used in the treatment of schizophrenia, acute manic or mixed episodes associated with bipolar disorder 1, and irritability associated with autistic disorder.^{1,2} The major metabolite of risperidone, 9-hydroxyrisperidone, is also pharmaceutically active. The therapeutic effect of risperidone is due to the total exposure to both risperidone and the active metabolite (total risperidone).³ The MyCare Total Risperidone Assay Kit measures the

total active risperidone in patient serum: risperidone plus 9-hydroxyrisperidone.

Nonadherence to medication is well known for patients with severe mental illness.⁴ While adherence to medication is critical to successful treatment outcomes, adherence is also least likely to be accurately assessed.^{5,6} Measurement of total risperidone provides clinicians with objective evidence of concentrations that may be related to patient adherence.⁷

The MyCare Total Risperidone Assay Kit (US Patent 8,088,594) is a homogenous two reagent nanoparticle agglutination assay used for detection of total risperidone (risperidone + 9-hydroxyrisperidone) in human serum. It is based on competition between drug and drug-conjugates for binding to drug specific antibodies covalently bound to nanoparticles. The extent of particle aggregation can be followed spectrophotometrically on clinical chemistry analyzers.

REAGENTS

| MyCare Total Risperidone Assay Kit REF RSP-RGT | Quantity x Volume |
|---|-------------------|
| Reagent 1 R1 Reaction buffer that contains drug-conjugate, protein and buffer | 1 x 10.0mL |
| Reagent 2 R2 Nanoparticle reagent that contains monoclonal antibody bound to nanoparticles in a buffered solution | 1 x 5.0 mL |

WARNINGS AND PRECAUTIONS

- For In Vitro Diagnostic Use Only.
- For diagnostic purposes, the results should always be assessed with the patient's medical history, clinical examination and other findings.
- Exercise normal precautions required for handling all laboratory reagents.
- All components of the MyCare Total Risperidone Assay Kit contain less than 0.1% sodium azide. Avoid contact with skin and mucous membranes. Flush affected areas with copious amounts of water. Seek immediate medical attention if reagents are ingested or come into contact with eyes. When disposing of such reagents, always flush with large amounts of water to prevent accumulation of azide.

REAGENT HANDLING

MyCare Total Risperidone Assay Kit reagents are ready to use. Before use, mix the reagents by gently inverting three to five times, avoiding the formation of bubbles.

STORAGE AND STABILITY

Store reagents refrigerated at 2-8°C (35-46°F). Do not freeze.

When stored and handled as directed unopened reagents are stable until the expiration date. Improper storage of reagents can affect assay performance.

SPECIMEN COLLECTION AND HANDLING

Serum is required. Trough or C_{min} samples at steady state have been recommended for testing antipsychotics.^{7,8} After one week of treatment on the same dose, collect samples 20 – 24 hours (daily dosing) or 9-12 hours (twice daily dosing) after the last dose.^{9,10} For long lasting injectables collect the sample before the next dose.¹¹

Prepare serum within 3 days of blood collection. Blood and serum samples may be stored at room temperature or 2-8°C. Store serum for up to 7 days before measuring. Freeze ($\leq 20^{\circ}\text{C}$) for longer storage. Avoid repeated freezing and thawing of samples.

PROCEDURE

Materials Provided:

REF RSP-RGT – MyCare Total Risperidone Assay Kit

Materials Required – Provided Separately:

REF MCP1-CAL – MyCare Psychiatry Calibrator Kit 1

REF MCP1-CON - MyCare Psychiatry Control Kit 1

Instruments

Reagents may need to be transferred to analyzer-specific reagent containers.

The performance of applications not validated by Saladax Biomedical, Inc. is not warranted and must be user defined.

Assay

To run the assay, see the instrument specific application sheet and appropriate analyzer operator's manual.

Calibration

Perform a full calibration using the six calibrators in the MyCare Psychiatry Calibrator Kit 1. Verify the calibration by testing the low, medium, and high controls in the MyCare Psychiatry Control Kit 1.

Calibration Frequency - Calibration is recommended:

- After a calibrator or reagent (kit) lot change,
- After performance of major instrument maintenance,
- As required following quality control procedures.

Quality Control (QC)

Each laboratory should establish its own QC procedures for the MyCare Total Risperidone Assay Kit. All quality control requirements and testing should be performed in accordance with local, state and/or federal regulations or accreditation requirements. Good laboratory practice suggests that at least two QC concentrations be tested each day patient samples are measured, and each time calibration is performed. Ensure that the quality control results meet the acceptance criteria before reporting patient results.

RESULTS

The concentration result is automatically calculated from the non-linear calibration curve by the analyzer. Report results in ng/mL or nmol/L. The conversion factor from ng/mL is $2.44 \times \text{ng/mL} = \text{nmol/L}$.

LIMITATIONS OF THE PROCEDURE

The MyCare Total Risperidone Assay Kit has been validated for serum. Do not use serum separator tubes.

As with any assay utilizing mouse antibodies, the possibility exists for interference by human anti-mouse antibodies (HAMA) in the sample. Samples containing such antibodies can potentially produce erroneous risperidone results, which are inconsistent with the patient's clinical profile.

Haloperidol when tested at 25 ng/mL had a cross-reactivity in the assay of $\leq 28\%$. Fentanyl when tested at 100 ng/mL had a cross-reactivity of $\leq 9\%$. Trazodone tested at 6,000 ng/mL had a cross-reactivity of 1%. Therefore, high therapeutic concentrations of trazodone may cause a bias in results. Elevated levels of total risperidone may be seen in patients administered haloperidol, fentanyl or trazodone.

EXPECTED VALUES

The therapeutic range for total risperidone in serum is not fully established. A therapeutic range from 20 to 60 ng/mL has been proposed.⁷ Measured concentrations for adherent patients at steady-state are expected to be in the measuring range of the assay.¹² Therapeutic drug monitoring of total risperidone has been recommended because of high interpatient variability, unpredictable response, and the importance of adherence for successful therapy.⁷ The complexity of the clinical state, individual differences in sensitivity, and co-administered medications may contribute to different requirements for optimal total risperidone blood levels. Users should investigate the transferability of the expected values to their own patient population and if necessary determine their own reference range. For diagnostic purposes the test findings should always be assessed in conjunction with the patient's medical history, clinical examinations, and other findings. Clinicians should carefully monitor patients during therapy initiation and dose adjustments. It may be necessary to obtain multiple samples to determine expected variation of optimal (steady-state) concentrations for individual patients.

SPECIFIC PERFORMANCE DATA

Typical performance data for the MyCare Total Risperidone Assay Kit obtained on a Beckman Coulter® AU480 are shown below. Results obtained in individual laboratories may differ from these data.

Precision

Within-laboratory precision and repeatability were verified throughout the measuring range according to CLSI Guideline EP5-A3.¹³ Three MyCare Psychiatry Control Kit 1 controls, three risperidone spiked pools (Serum 1, 2, 3) and two pools of clinical samples (Clinical 1, 2) were tested.

| Sample | N | Mean (ng/mL) | Repeatability | Within-Laboratory |
|------------|----|--------------|---------------|-------------------|
| | | | CV | CV |
| Control 1 | 80 | 36 | 2.8% | 3.7% |
| Control 2 | 80 | 65 | 2.1% | 2.8% |
| Control 3 | 80 | 99 | 2.5% | 3.3% |
| Serum 1 | 80 | 21 | 3.3% | 5.0% |
| Serum 2 | 80 | 59 | 2.4% | 4.2% |
| Serum 3 | 80 | 78 | 3.3% | 6.0% |
| Clinical 1 | 80 | 22 | 3.0% | 4.2% |
| Clinical 2 | 80 | 58 | 3.1% | 3.8% |

Limit of Quantitation (LoQ) and Limit of Detection (LoD)

The lower limits of quantitation and detection were established using CLSI guideline EP17-A2.¹⁴

LoQ

The LoQ was determined with an accuracy goal at the LoQ of $\leq 35\%$ total error (Westgard model). The LoQ of the MyCare Total Risperidone Assay Kit is 16 ng/mL.

LoD

The LoD is the lowest amount of analyte that can be reliably detected (≥ 95 of results greater than the limit of blank.). The LoD of the MyCare Total Risperidone Assay Kit is 6 ng/mL.

Result Reporting

Each laboratory should determine reporting criteria for total risperidone concentrations. The following suggestion from CLSI EP17-A2 may be appropriate:¹⁴

Result \leq LoB - report "not detected; concentration < LoD"

LoB < Result < LoQ - report "analyte detected; concentration < LoQ"

Result \geq LoQ - report the result as measured

Measurement Range

The measurement range of the MyCare Total Risperidone Assay Kit is 16 – 120 ng/mL.

Specificity

Metabolism

Risperidone is extensively metabolized in the liver by CYP2D6 and to a lesser extent by CYP3A4.¹ The biotransformation by CYP2D6 gives the major metabolite (\pm) 9-hydroxy-risperidone, both enantiomers of which are as active as the parent drug. The therapeutic effect of risperidone is due to the total exposure to both risperidone and the active metabolite.

There are two minor metabolites in serum. 7-hydroxyrisperidone occurs as 1-5% of parent drug.¹⁵ The minor N-desalkyl-risperidone metabolite has been reported to occur at 10 – 13% of the parent drug.¹⁵

Specificity for the following metabolites and cross-reactants was tested in the absence and presence of risperidone at 20 and 60 ng/mL.

Risperidone metabolites

| Compound | Tested at (ng/mL) | Cross-Reactivity |
|------------------------|---|------------------|
| 9-hydroxyrisperidone | with risperidone for total risperidone concentrations of 20, 60 and 120 ng/mL | 101% |
| 7-hydroxyrisperidone | 10 | <60% |
| N-desalkyl risperidone | 20 | < 5% |

Cross-reactivity

The following compounds did not interfere with the MyCare Total Risperidone Assay Kit: cross reactivity was $\leq 3\%$ or the assay bias was $\leq 10\%$.

| Compound | Tested at (ng/mL) | Compound | Tested at (ng/mL) |
|----------------------|-------------------|--------------------|-------------------|
| Acetaminophen | 200,000 | Acetazolamide | 60,000 |
| Acetylsalicylic acid | 500,000 | Albuterol | 1,000 |
| Alendronate sodium | 1,000 | Alpha - tocopherol | 40,000 |

| Compound | Tested at (ng/mL) | Compound | Tested at (ng/mL) |
|----------------------------------|-------------------|-------------------------|-------------------|
| Alprazolam | 2,000 | Amantadine | 10,000 |
| Amikacin sulfate | 100,000 | Amiloride HCl dihydrate | 500 |
| Amisulpride | 400 | Amitriptyline | 1,000 |
| Amlodipine besylate | 100 | Amoxicillin | 80,000 |
| S (+)-amphetamine | 1,000 | Aripiprazole | 500 |
| L-ascorbic acid | 60,000 | Atomoxetine | 5,000 |
| Atorvastatin calcium | 600 | Baclofen | 3,000 |
| Benzotropine | 400 | Betamethasone | 100 |
| Biotin | 300 | Biperiden | 100 |
| Budesonide | 2.2 | Bupropion | 3,000 |
| Buspirone | 20 | Caffeine | 60,000 |
| Calcium carbonate | 300,000 | Carbamazepine | 30,000 |
| Cefalexin | 200,000 | Celecoxib | 1,000 |
| Cetirizine dihydrochloride | 3,500 | 8-chlorotheophylline | 3,000 |
| Chlorpromazine HCl | 2,500 | Cimetidine | 20,000 |
| Ciprofloxacin | 10,000 | Citalopram HBr | 750 |
| Clindamycin | 50,000 | Clonazepam | 150 |
| Clotrimazole | 50 | Clozapine | 1,000 |
| Codeine | 2,000 | Cortisol | 300 |
| (-)-cotinine | 2,000 | Desloratadine | 600 |
| Desvenlafaxine | 400 | Dextromethorphan | 1,000 |
| Diazepam | 6,000 | Diphenhydramine HCl | 6,000 |
| Docosahexaenoic acid ethyl ester | 150,000 | Doxycycline HCl | 35,000 |
| Duloxetine | 200 | Erythromycin | 60,000 |
| Escitalopram | 100 | Estradiol | 1.2 |
| Eszopiclone | 200 | Ethanol | 4,000,000 |
| Famotidine | 600 | Fenofibrate | 50,000 |
| Fluoxetine HCl | 4,000 | Fluticasone propionate | 1 |
| Folic acid | 15 | Furosemide | 60,000 |
| Gentamycin sulfate | 30,000 | Glyburide | 2,000 |
| Haloperidol decanoate | 1,500 | Heparin sodium salt | 3 U/mL |
| Hydrochlorothiazide | 6,000 | Ibuprofen | 500,000 |
| lloperidone | 10 | Indinavir sulfate | 400 |
| Lamivudine | 2,000 | Lamotrigine | 15,000 |

| Compound | Tested at (ng/mL) | Compound | Tested at (ng/mL) |
|-------------------------|-------------------|-----------------------------|-------------------|
| Lansoprazole | 1,000 | Lisinopril dihydrate | 350 |
| Lithium carbonate | 250,000 | Lorazepam | 1,000 |
| Lovastatin | 500 | Meclizine | 500 |
| Metformin | 40,000 | Methotrimeprazine | 200 |
| Methylphenidate HCl | 350 | Metoclopramide HCl | 500 |
| Metoprolol tartrate | 5,000 | Metronidazole | 120,000 |
| Mirtazapine | 300 | Mometasone furoate | 1 |
| Morphine | 500 | Naproxen sodium | 500,000 |
| Nateglinide | 20,000 | N-desalkyl risperidone | 20 |
| Nefazodone HCl | 3,500 | Nicotinic acid | 20,000 |
| Nordiazepam | 5,000 | Nortriptyline | 1,000 |
| Olanzapine | 300 | Omeprazole | 6,000 |
| Oxazepam | 5,000 | Oxcarbazepine | 35,000 |
| Oxycodone | 500 | Pantothenic acid | 150 |
| Paroxetine | 1,000 | Penicillin V | 6,000 |
| Perphenazine | 100 | Phentermine | 500 |
| Pimozide | 20 | Pipamperone dihydrochloride | 400 |
| Potassium EDTA | 1,000 | Pravastatin sodium | 150 |
| Prednisolone | 3,000 | Pregabalin | 5,000 |
| Promethazine | 1,200 | R,R (-)-pseudoephedrine | 10,000 |
| S,S (-)-pseudoephedrine | 10,000 | Pyridoxine HCl | 100 |
| Quetiapine | 500 | Quinidine | 12,000 |
| Ranitidine | 6,000 | Retinol | 4,000 |
| Riboflavin | 200 | Rosuvastatin calcium | 50 |
| Salicylic acid | 500,000 | Sertraline hydrochloride | 600 |
| Simvastatin | 30 | Sodium fluoride | 150 |
| Spironolactone | 600 | Sulfamethoxazole | 400,000 |
| Temazepam | 5,000 | Theophylline | 40,000 |
| Thiamine HCl | 50 | Topiramate | 10,000 |
| Triamcinolone acetonide | 10 | Triamterene | 9,000 |
| Triazolam | 40 | Valproic acid | 500,000 |
| Vancomycin HCl | 100,000 | Venlafaxine HCl | 400 |
| Vitamin B12 | 1 | Vitamin D2 | 40 |

| Compound | Tested at (ng/mL) | Compound | Tested at (ng/mL) |
|-------------|-------------------|-----------------------|-------------------|
| Vitamin K1 | 1 | Warfarin | 10,000 |
| Ziprasidone | 200 | Zolpidem hemitartrate | 5,000 |
| Zonisamide | 40,000 | Zopiclone | 100 |

Interfering Substances

No significant assay bias was observed from samples with the following endogenous interferents at the given levels.

| Interferent | Level | |
|-----------------------------|-------------|------------|
| Rheumatoid Factor | 508 IU/mL | |
| Total Protein Matrix Effect | 11.0 g/dL | 110 g/L |
| Icteric Interference | 18.32 mg/dL | 313 µmol/L |
| Lipemic Interference | 1,828 mg/dL | 20 mmol/L |
| Hemolysate | 210 mg/dL | |

Recovery

Patients on risperidone therapy have both risperidone (RSP) and the active metabolite 9-hydroxyrisperidone (9-OH RSP) in their serum. Therefore, to assess recovery of the MyCare Total Risperidone Assay Kit, risperidone and the active metabolite 9-hydroxyrisperidone were spiked together into four individual normal risperidone-free sera. The percent recovery was determined by dividing the observed concentration of each sample by the expected concentration of added risperidone plus 9-hydroxyrisperidone.

Mean Percent Recovery

| Theoretical ng/mL | RSP:9-OH RSP ratio | Percent Recovery | RSP:9-OH RSP ratio | Percent Recovery |
|-------------------|--------------------|------------------|--------------------|------------------|
| 20 | 4:1 | 90 – 120 | 1:4 | 90 – 120 |
| 60 | 4:1 | 90 - 108 | 1:4 | 92 – 115 |
| 120 | 4:1 | 90 – 110 | 1:4 | 95 - 115 |

Linearity

The linearity of the MyCare Total Risperidone Assay Kit was verified according to CLSI guideline EP6-A.¹³ Eleven linearity samples covering the measuring range were prepared in human serum spiked with risperidone. Deviation from linearity (n=5) was < 4%. The assay was linear across the measuring range from 16 – 120 ng/mL.

Method Comparison

Results of the MyCare Total Risperidone Assay Kit were compared to a validated LC-MS/MS according to CLSI guideline EP09-A3.¹⁷ Deming regression analysis was performed with 146 patient samples. Results are shown for one lot.

| Deming Regression statistics MyCare Total Risperidone Assay Kit vs. LC-MS/MS | |
|---|----------------|
| Slope | 0.98 |
| Intercept | 1.23 |
| Correlation Coefficient (R) | 0.96 |
| N | 146 |
| Concentration Range (LC-MS/MS) | 16 – 118 ng/mL |

References

1. Janssen Pharmaceuticals, Inc. Risperdal® (risperidone) Prescribing information. 2017.
2. Janssen Pharmaceuticals, Inc. Risperdal Consta® (risperidone) Long-acting Injection Prescribing Information. 2017.
3. Seto K, Dumontet J, Ensom MH. Risperidone in schizophrenia: is there a role for therapeutic drug monitoring? *Ther Drug Monit.* 2011;33(3):275-283.
4. Velligan DI, Weiden PJ, Sajatovic M, et al. Assessment of adherence problems in patients with serious and persistent mental illness: recommendations from the Expert Consensus Guidelines. *J Psychiatr Pract.* 2010;16(1):34-45.
5. Higashi K, Medic G, Littlewood KJ, Diez T, Granstrom O, De Hert M. Medication adherence in schizophrenia: factors influencing adherence and consequences of nonadherence, a systematic literature review. *Ther Adv Psychopharmacol.* 2013;3(4):200-218.
6. Haddad PM, Brain C, Scott J. Nonadherence with antipsychotic medication in schizophrenia: challenges and management strategies. *Patient Relat Outcome Meas.* 2014;5:43-62.
7. Hiemke C, Bergemann N, Clement HW, et al. Consensus Guidelines for Therapeutic Drug Monitoring in Neuropsychopharmacology: Update 2017. *Pharmacopsychiatry.* 2018;51:9-62.
8. Bowskill SVJ, Handley SA, Fisher DS, Flanagan RJ, Patel MX. Risperidone and Total 9-Hydroxyrisperidone in Relation to Prescribed Dose and Other Factors: Data From a Therapeutic Drug Monitoring Service, 2002–2010. *Ther Drug Monit.* 2012;34(3):349-355.
9. Lostia AM, Mazzarini L, Pacchiarotti I, et al. Serum levels of risperidone and its metabolite, 9-hydroxyrisperidone: correlation between drug concentration and clinical response. *Ther Drug Monit.* 2009;31(4):475-481.
10. Aravagiri M, Marder SR, Nuechterlein KH, Gitlin MJ. Intra- and interindividual variations in steady-state plasma concentrations of risperidone and 9-hydroxyrisperidone in schizophrenic patients treated chronically with various doses of risperidone. *Ther Drug Monit.* 2003;25(6):657-664.
11. Gefvert O, Eriksson B, Persson P, et al. Pharmacokinetics and D2 receptor occupancy of long-acting injectable risperidone (Risperdal Consta) in patients with schizophrenia. *Int J Neuropsychopharmacol.* 2005;8(1):27-36.
12. Melkote R, Singh A, Vermeulen A, Remmerie BM, Savitz A. Relationship between Antipsychotic Blood Levels and Treatment Failure during the CATIE Study. International Congress on Schizophrenia Research. 2017;Poster.
13. CLSI. Evaluation of Precision Performance of Quantitative Measurement Methods; Approved Guideline-Second Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute, 2014.
14. CLSI. Evaluation of Detection Capability for Clinical Laboratory Measurement Procedures; Approved Guideline – Second Edition. CLSI document EP17-A2. Wayne, PA: Clinical and Laboratory Standards Institute; 2012.
15. Mannens G, Huang ML, Meuldermans W, Hendrickx J, Woestenborghs R, Heykants J. Absorption, metabolism, and excretion of risperidone in humans. *Drug Metab Dispos.* 1993;21(6):1134-1141.
16. NCCLS. Evaluation of the Linearity of Quantitative Measurement Procedures: A Statistical Approach; Approved Guideline. NCCLS document EP6-A. Wayne, PA: NCCLS; 2003.
17. CLSI. Measurement Procedure and Bias Estimation Using Patient Samples; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.

© 2018, Saladax Biomedical, Inc.

MyCare™ is a trademark of Saladax Biomedical, Inc. All other product names and trademarks are property of their respective owners.