

APPLICATION SHEET - Roche cobas® pro - c 503

MyCare™ Psychiatry Clozapine Assay

Reagent ID: CLZ-RGT

Clozapine

Configure analytical parameters					
Short Name	CLZ		Unit 1	ng/ml	
Long Name	SBI Clozapine		Unit 2	None	
Rerun Setting			Conversion Factor	1	
Below	OFF				
Above	OFF				
Technical Limit	68	-	1500		
Assay					
Assay	2 Point End		Primary	600	
Time	10		Secondary	NA	
Points	1	20	2	34	
	3	0	4	0	
Cassette Configuration					
	Reagent Volume	Dilution Volume	Dummy Reagents	Mode	Mix
R1	70	0	0	Without	4
R2	0	0	0	Without	4
R3	35	0	0	Without	4
Sample Volume					
	Sample Volume	Diluted Sample Vol.	Diluent Volume	Mix	
Normal	2.8	0	0	4	
Decreased	2.8	0	0	4	
Increased	2.8	0	0	4	
Diluent	Type	Water			
	ACN	None			
	Dilution	0			
Cell Wash					
Wash Cells With	Basic and Acid				
Higher Uncertainty	85				
QC Interval					
QC Interval Timeout	Disabled				

APPLICATION SHEET - Roche cobas® pro - c 503

MyCare™ Psychiatry Clozapine Assay

Reagent ID: CLZ-RGT

Clozapine

Configure R.P. settings			
c pack	***		
No. of Tests	105		
Position	Reagent Type	Filling Volume	
B	R1	12	
C	R3	6.6	
*** Select from drop down; reagent packs CDC01–CDC10 are dedicated to development channel			
Calculate Volume			

Configure Ranges			
Linearity Limit			
4-8 Point	0		
9 Point	0		
Min. Total Rate	0		
Min. Diff. Rate	0		
Reaction Limit			
Check	Off		
Abs. Limit	0		
Application Correction Factor			Method Decrease
	A	1.0	Sample Index Limits
	B	0.00	L 0
			H 0
			I 0



APPLICATION SHEET - Roche cobas® pro - c 503

MyCare™ Psychiatry Clozapine Assay

Reagent ID: CLZ-RGT

Clozapine

Configure calibration parameters						
Changeover Settings						
Lot Changeover	Full					
Reagent Pack Changeover	Cancel					
Calibration Trigger						
QC Violation	Method Cancel					
Timeout	Method Full		Stability 21			
Limit Values						
SD Limit	0					
Duplicate Limit	99 % 3.3 Abs.					
Sensitivity Start	0					
Sensitivity End	0					
Sensitivity Limit	-9.9999		9.9999			
S1 Abs. Limit	-3.3000		3.3000			
Calibration Method						
Curve Type	RCM1	Point	6	Weighting	0	
Configuring calibrators						
	S1	S2	S3	S4	S5	S6
Calibrator Code	*	*	*	*	*	*
Calibrator Volume	2.8	2.8	2.8	2.8	2.8	2.8
Diluted C. Volume	0	0	0	0	0	0
Diluent Volume	0	0	0	0	0	0
Calibrator Diluent						
Type	Water					
Code	None					
Dilution Factor	0					
*Calibrator Codes for development channels are 29001 - 29050						



APPLICATION SHEET - Roche cobas® pro - c 503

MyCare™ Psychiatry Clozapine Assay

Reagent ID: CLZ-RGT

Clozapine

Configure assay parameters – Calibration / Install / Chemistry							
Define the following values by choosing EDIT for EACH MCP2 calibrator A through F.							
Edit Calibrator							
Name:	CLZ	Code:	**	Lot No.:	Cal Kit Lot		
Test	Std (1)	Std (2)	Std (3)	Std (4)	Std (5)	Std (6)	
CLZ	0	75	121	248	539	1423	
							Std Concentration
							Std (1) 0
							Std (2) 75
							Std (3) 121
							Std (4) 248
							Std (5) 539
							Std (6) 1423
							Unit of Measure ng/ml
** Calibrator Codes for development channels are 29001 - 29050							

For further information refer to the package insert or

Contact Saladax Biomedical Customer Support at: Telephone: +1 (610) 419 - 6731
E-mail: techsupport@saladax.com