### MyPaclitaxel™ Control Kit for cobas® c 111

This package insert must be read carefully prior to product use. Package insert instructions must be followed accordingly. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.


**Customer Service:**

Telephone: +1-610-419-6731

Fax: +1-484-547-0590

#### Key to Symbols Used

	<i>in vitro</i> Diagnostic Device		Temperature Limitation						
	Batch Code (Lot)		Use By						
	Catalog Number		Authorized Representative in the European Community						
	Caution: Consult accompanying documents		Manufacturer						
	Consult Instructions for Use	<table border="1"> <tr> <td>CON</td> <td>L</td> </tr> <tr> <td>CON</td> <td>M</td> </tr> <tr> <td>CON</td> <td>H</td> </tr> </table>	CON	L	CON	M	CON	H	Low Control Medium Control High Control
CON	L								
CON	M								
CON	H								

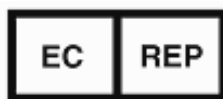EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Saladax Biomedical, Inc.  
116 Research Drive  
Bethlehem, PA 18015 USA  
www.saladax.com

## Intended Use

The Saladax MyPaclitaxel Control Kit is intended for use in quality control of the Saladax MyPaclitaxel Assay.

## Contents

The MyPaclitaxel Control Kit consists of buffer matrix with the following concentrations of paclitaxel:

Vial		Target Concentration (ng/mL)	Expected Range of Results (ng/mL)	Quantity	Volume
CON	L	50	43-58	2	3 mL
CON	M	110	94-127	2	3 mL
CON	H	190	162-219	2	3 mL

When tested with the Saladax MyPaclitaxel Assay, the control results are expected to be within 15% of the target concentration. Each laboratory should establish its own values and reassess control targets and ranges following a change of reagent (kit) or control lot.

## Precautions and Warnings

For In Vitro Diagnostic Use.

The controls in this set are designed for use as a unit. Do not substitute or mix controls with those from other lots.

Exercise the normal precautions required for handling all laboratory reagents.

All MyPaclitaxel Controls contain less than 0.1% sodium azide. Avoid contact with skin and mucous membranes. Flush affected areas with copious amounts of water. Seek immediate medical attention if controls are ingested or come into contact with eyes.

## Storage and Stability

Store controls refrigerated at 2-8°C (35-46°F). Do not freeze. Controls are stable until the expiration date when stored as directed. Once opened controls are stable for 2 months when handled as directed.

## Limitations of the Procedure

Accurate and reproducible results are dependent upon properly functioning instruments, reagents, calibrators, correct product storage, and good laboratory technique.

## Instructions for Use

- Refer to the Saladax MyPaclitaxel Assay package insert, included in the reagent kit, for a complete summary and explanation of the test.
- Use controls immediately upon removal from 2-8°C storage.
- Mix each control by gentle inversion several times before dispensing.
- After each use, tightly close the caps and return control to 2-8°C.

Bubbles may interfere with proper detection of control level in the sample cup, causing insufficient control aspiration that could impact results.

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